
economy visionLED

Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso
Manual de instrucciones
Gebruiksaanwijzing
Betjeningsvejledning
Bruksanvisning
Käyttöopas
Instrukcja obsługi
Provozní návod

ESCHENBACH

Deutsch	4
English.....	6
Français	8
Italiano.....	10
Español.....	12
Nederlands	14
Dansk	16
Svenska.....	18
Norsk.....	20
Suomi.....	22
Polski.....	24
Česki	26

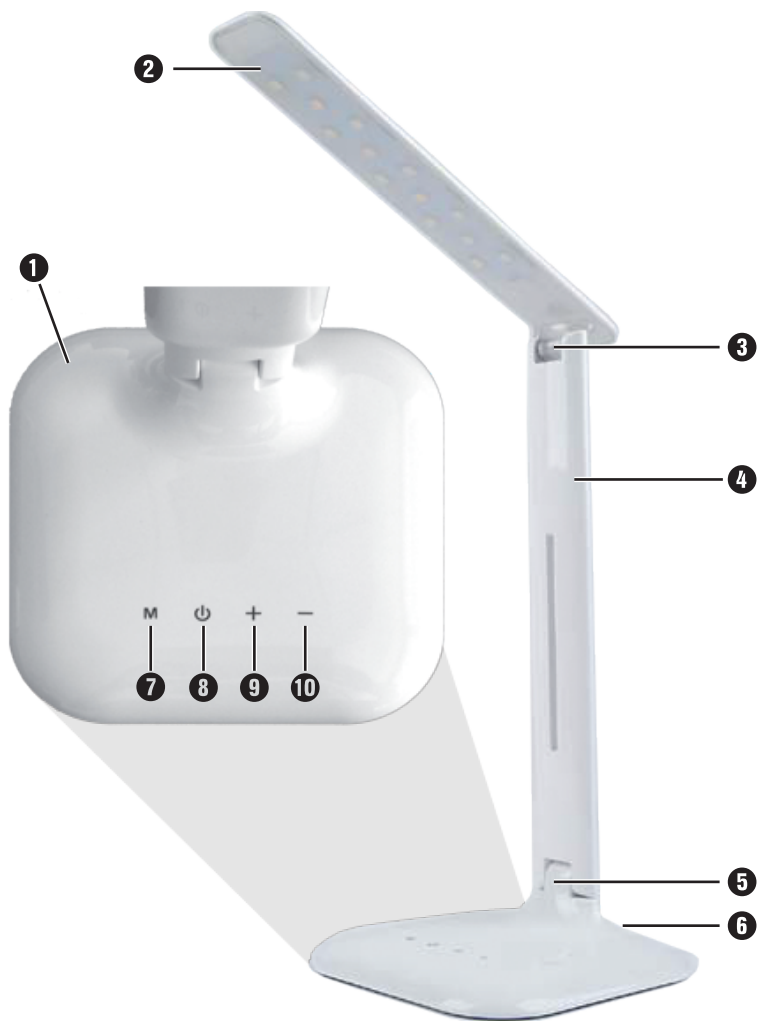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Safety instructions

- ▶ Risk of dazzling and injury! Do not look at the light source for long periods!
- ▶ Protect your desk lamp from impact or shock, moisture and excessive heat!
Never place your desk lamp near a heat source or in direct sunlight.
- ▶ Do not use the desk lamp near containers filled with water!
- ▶ Risk of tripping! Make sure that the power cord is not posing a danger!
Do not wrap the power cord around the table lamp!
- ▶ Ensure that other people, particularly children, are aware of these risks!

Operating elements

- ❶ Stand
- ❷ Lamp head
- ❸ Arm for adjusting the lamp head
- ❹ Support arm for the lamp head
- ❺ Joint to adjust the support arm for the lamp head
- ❻ Socket for the power cord plug
- ❼ Color temperature toggle switch M
- ❽  button: switching on and off
- ❾ + button: increase brightness
- ❿ - button: decrease brightness

Starting up

1. Set the desk lamp on a flat and stable surface.
2. Plug the connector of the supplied power cord into the socket ❻ at the rear of the lamp base.
3. Insert the mains plug of the cable into a properly wired electrical outlet.
4. Gently tap the touch button  ❽ to turn the lamp on or off. A slight warming in the area of lit LEDs is normal.
5. Use the touch buttons + ❾ and - ❿ to adjust the brightness. Repeatedly tap briefly + ❾ to increase the brightness in 5 steps. Repeatedly tap briefly - ❿ to decrease the brightness in 5 steps.
6. Tap on the toggle switch M ❼ repeatedly to switch through the 3 available color temperatures:
2800 K, provides a yellowish warm-white light.
3850 K, provides a yellowish-white light.
4900 K, provides a slightly blue-white light.
7. Adjust the lamp head ❷ and its support arm ❸ to suit your requirements.
8. To disconnect the unit from the mains power supply, unplug the power cord from the wall outlet.

Care instructions

Unplug the desk lamp from the power supply before cleaning!

Clean the device with a damp cloth. Do not use soap solutions containing plasticisers, alcoholic solvents or abrasive cleaners!

Technical specifications

Power supply voltage	AC 110 V – 230 V ~		
Illuminance (in 25 cm)	2800 K	3850 K	4900 K
Level 1	359 lx	1175 lx	880 lx
Level 2	285 lx	928 lx	685 lx
Level 3	213 lx	680 lx	500 lx
Level 4	145 lx	450 lx	342 lx
Level 5	75 lx	228 lx	172 lx
Power consumption	approx. 4 W		

Warranty

Within the parameters of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing defects or material defects. In event of damage due to improper handling, including damage from a fall or impact, no warranty claims can be made.

Warranty claims are only accepted upon presentation of the sales receipt!

Disposal

When the device reaches the end of its useful life, do not dispose of it with the normal household waste. This product is subject to the European directive in its currently valid version.

Dispose of the product or parts of the product through a certified disposal facility or your municipal disposal facility. Observe the currently applicable regulations. In case of doubt, contact your disposal facility. Dispose of all packaging materials in an environmentally friendly manner.

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Eschenbach Optik GmbH